F

PROPRIÉTÉS

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UK

PROPERTIES

- Paste for temporary filling,
 - adheres to dentine,
 - hardens rapidly in the cavity,
 - expands slightly while setting,
 - ensures tight sealing.

WARNINGS AND PRECAUTIONS FOR USE

Like any filling material, CIMPAT remains in place only when the cavity is properly shaped. CIMPAT may be softened during setting by certain solvents contained in liquids or pastes used for root canal treatment. Therefore after any root canal treatment and before filling with CIMPAT, it is required to thoroughly clean the cavity walls and allow them to dry.

For easy handling, use a warm spatula.

T – Toxic
 N – Dangerous for the environment
 Contains dibutyl phtalate (DBP)

Very toxic to aquatic organisms, may cause long-term adverse effects in the aquatic environment.

Irritating to eyes.

May cause harm to the unborn child.

Possible risk of impaired fertility.

Wear suitable protective clothing and gloves.

In case of insufficient ventilation, wear suitable respiratory equipment.

In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).

Avoid exposure - obtain special instructions before use.

Avoid release to the environment. Refer to special instructions/safety data sheets.

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