F

**COMPOSITION**

Fil de coton.

**INDICATION**

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Cotton cord

**INDICATION**

Retraction of the free gingiva prior to impression taking.

**DIRECTIONS FOR USE**

Septofil produces a mechanical retraction. It temporarily opens the sulcus, so as to take impressions or carry out some dental procedures. If gingival retraction is needed for several teeth, treat one tooth at a time.

Pull the cord out of the bottle and cut the needed length: the peri-

meter of the crown portion of a tooth can vary from 2 to 6 or even 7 cm. Cut longer segments of cord than required so that when the cord is wound up around the tooth, the ends overlap. This makes the removal of the cord easier.

Place the cord in the sulcus (gingival crevice) using a metallic blunt instrument so as not to injure the epithelial attachment. Start with the interproximal space because it is where the sulcus is usually the deepest. Then, proceed clockwise until you have gone all the way around the tooth.

To obtain a greater retraction, once the cord has been inserted, it may be impregnated with a solution for retraction. Leave it in situ for the time indicated in the directions for use of the solution. The cord should be removed just before taking impressions except if the double impression technique is used. Slightly moisten the cord to make its removal easier and less painful.

**WARNINGS AND PRECAUTIONS FOR USE**

- After impression-taking, the gingival sulcus should be inspected thoroughly to make sure that the epithelial attachment has not been torn and that no fragment of the retraction cord has been left in the gingival crevice, which would act as a foreign body and induce a severe local inflammation.
- This cord should only be used on healthy tissues, free from injury, bleeding and infection.

**STORAGE**

Store the product away from moisture.

**PRESENTATIONS**

- Bottle, with a dispensing stopper, containing 2.5 metres of ultra-thin cord.
- Bottle, with a dispensing stopper, containing 2.5 metres of thin cord.
- Bottle, with a dispensing stopper, containing 2.5 metres of medium cord.
- Bottle, with a dispensing stopper, containing 2.5 metres of thick cord.
- Box of 4 bottles, each of them containing 2.5 m of cord of assorted diameters (ultra-thin, thin, medium and thick)

*For professional use only in the practice of dentistry.*

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