**SeptoDiamond Paste**  (EN)

Diamond polishing paste

Diamond-based polishing paste with extra-fine grit, indicated for general use in the dental office and laboratory. When applied with the aid of felt discs, SeptoDiamond Paste provides an excellent gloss on ceramic, composite, amalgam, and metal surfaces. The paste vehicle, which is composed of water-soluble ingredients, facilitates its removal at the end of the procedure.

**Use:** SeptoDiamond Paste should be used with felt discs to enhance its action on providing the final polishing. Prepare the area to be polished, by modeling it and finishing it to achieve the desired anatomy and a smooth surface. Fit the disc into the mandril. Apply the paste on the disc, or directly on the tooth or the restorative material. Apply the product using a low speed and applying low pressure, with intermittent strokes and circular movements. After the procedure, it is recommended to rinse with water jet and suction.

**Warnings, precautions and restrictions:** Professional use. For single use only. Reuse may cause cross-contamination. The product should not be applied in patients allergic to any of its components. Wear protective goggles to shield against particles (dentists and patients).

Wear a surgical mask to avoid inhalation of particles. Follow the instructions for use to prevent excessive heating of the surface. Keep the product tightly sealed in its original packaging. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**Presentacion:** 0101060003 - SeptoDiamond Paste - 2g

**Composicion:** Polietilene glicol, diamond and flavoring agent .

**Storage:** To be stored between 15 °C and 30 °C (59 °F and 86 °F), in dry place and protected from sunlight.

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