

SeptoAccessories

From now. Everyday. Septodont Accessories.



Index

INSTRUMENTS

Septo Contact.....	4
Septo Compo Shape	6
Septo Cure.....	8

FINISHING AND POLISHING

Septo Discs	10
Septo Wheels PX	12
Septo Polisher.....	14
Septo Fipo Strip.....	16
Septo Diamond Strip	18
Septo Diamond Paste	20

MATRICES

Septo Matrix Cervical	22
Septo Matrix Sectional	24

WEDGES

Septo Wedges	26
---------------------------	----

ISOLATION

Septo Tape.....	28
------------------------	----

PROPHYLAXIS

Septo Prophy Strip	30
---------------------------------	----

INSTRUMENTS

SeptoContact

INSTRUMENTS FOR CONTACT POINT RECONSTRUCTION

INDICATIONS

To a large extent instruments determine the functional properties of dental materials to a large extent. Well-chosen instrumentation influences the effectiveness and ergonomics of dentists' work. SeptoContact is a very practical instrument designed to easily reconstruct consistent and precise contact points in composite restoration procedures.

FEATURES AND BENEFITS

The guarantee of reconstructing a perfect contact point in a repeatable "three step" protocol.

- One instrument for large cavities (L) and one for small cavities (S).
- Each instrument has a distal and a mesial tip.
- Translucent instruments allow for light curing; suitable for sterilisation.



PACKAGING

DESCRIPTION

SeptoContact:

2 instruments, each with a distal and mesial tip

SeptoContact
Instruments



INSTRUMENTS

SeptoCompo Shape


SILICONE BRUSHES FOR COMPOSITE SHAPING

INDICATIONS

Silicone brushes, SeptoCompo Shape, are a great idea for precise and successful modeling of direct and indirect restorations. Ease of manipulation, control of adaptation of subsequent layers and the ability to control each composite in terms of its adhesion are the undisputed advantages of these instruments. Each movement with SeptoCompo Shape easily forms the composite into the ideal, anatomical shape of the filling.

FEATURES AND BENEFITS

- Allows the avoidance of lines and marks between the composite layers caused by the layering technique
- Form smooth surfaces with a significant time reduction for posterior finishing and polishing
- Available in two versions, with hard and soft tips, for optimal results regardless of the composite in use
- Tips available in 4 different shapes, which allows their application to a wide range of clinical situations
- SeptoCompo Shape is suitable for sterilisation, together with the silicone tips.

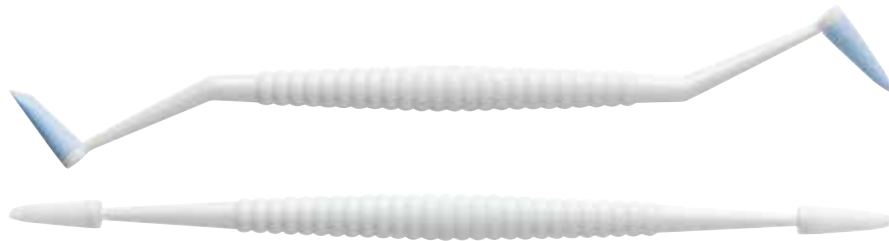

PACKAGING

DESCRIPTION

SeptoCompo Shape:

2 instruments + 8 silicone tips (soft/hard)

SeptoCompo Shape Refill: 4 soft silicone tips

SeptoCompo Shape Refill: 4 hard silicone tips



SeptoCompo Shape
Instruments



INSTRUMENTS

SeptoCure

INSTRUMENTS WHICH REDUCE THE RISK OF MARGINAL INTEGRATION OF DIRECT ADHESIVE RESTORATIONS IN THE GINGIVAL REGION

INDICATIONS

SeptoCure is specially designed to improve marginal adaptation and increase efficiency of polymerisation of dental composites in areas with difficult direct access. Leakage of Class II fillings in the area of gingival contact walls will be substantially reduced once the dentist uses an instrument that ideally conducts the light from the polymerisation unit and improves adaptation of the composite. Thanks to SeptoCure the direct composite filling and bonding system will be fully cured, which means not only excellent marginal integrity, but also optimised mechanical properties.

FEATURES AND BENEFITS

- Developed to prevent and minimise marginal leakage of resins in Class II restorations
- Does not adhere to the composite during polymerisation
- Translucent instrument enables more effective composite curing by means of light transmission through the instrument's tip
- Enables proper composite adaptation to the base of the cavity
- Suitable for sterilisation.



PACKAGING

DESCRIPTION

SeptoCure: 2 instruments, 4 different tips

SeptoCure
Instruments



FINISHING AND POLISHING

SeptoDiscs

FINISHING AND POLISHING DISCS FOR COMPOSITES

INDICATIONS

SeptoDiscs allow the most important details of the surface to be achieved: smoothness and gloss!

The best range of flexibility: allows easy access to all proximal and interproximal surfaces.

Reduced thickness: facilitates access to difficult areas and simply enables contouring and finishing of proximal surfaces.

High quality final effect: minimal surface roughness, high gloss.

High abrasiveness: better final finishing results in a shorter time, easy removal of all excess composite.



PACKAGING

DESCRIPTION

SeptoDiscs Kit: 120 discs + SeptoDiscs Mandrel
(ř 1/2": 20x Coarse, 20x Medium, 20x Fine, 20x Extra-Fine,
ř 3/8": 10x Coarse, 10x Medium, 10x Fine, 10x Extra-Fine)

SeptoDiscs Assorted Refill ř 1/2": 50 discs
(15x Coarse, 15x Medium, 10x Fine, 10x Extra-Fine)

SeptoDiscs Coarse Refill ř 1/2": 50 discs

SeptoDiscs Medium Refill ř 1/2": 50 discs

SeptoDiscs Fine Refill ř 1/2": 50 discs

SeptoDiscs Extra-Fine Refill ř 1/2": 50 discs

FEATURES AND BENEFITS

- Handy, durable and easy application
- Pop-on mandrel system, easy and fast installation of the discs as well as the inversion of the abrasive surface for mesial and distal positions
- Ultra-fine discs are very flexible and versatile. Ideal for difficult to access areas.
- Four colour coded grits, available in 1/2" and 3/8" sizes



DESCRIPTION

SeptoDiscs Mandrel: 1 pcs.

SeptoDiscs Assorted Refill ř 3/8": 50 abrasive discs

SeptoDiscs Coarse Refill ř 3/8": 50 abrasive discs

SeptoDiscs Medium Refill ř 3/8": 50 abrasive discs

SeptoDiscs Fine Refill ř 3/8": 50 abrasive discs

SeptoDiscs Extra Fine Refill ř 3/8": 50 abrasive discs

SeptoDiscs

Finishing & Polishing



FINISHING AND POLISHING

SeptoWheels PX

IMPREGNATED FELT WHEELS

INDICATIONS

Performing the perfect filling requires skill, ideal materials and reliable instruments. To achieve a unique final effect of polishing, an indispensable element are the SeptoWheels PX felt discs!

Light reflecting from the surface gives the desired outcome of a natural looking reconstruction. SeptoWheels PX felt discs are composed of natural wool impregnated with abrasive polishing paste, activated with a few drops of water.

FEATURES AND BENEFITS

- Excellent effect during final polishing, i.e. naturally shiny surfaces.
- Provides beautiful, white and bright teeth after whitening procedures
- Impregnated with ultra-fine dehydrated polishing paste, activated with just a few drops of natural water Available in 9,5 mm size



PACKAGING

DESCRIPTION

SeptoWheels PX*:
12 impregnated felt wheels + SeptoPlus Mandrel

SeptoPlus Mandrel: 1 pcs.

*SeptoPlus Mandrel required

SeptoWheels PX

Finishing & Polishing



FINISHING AND POLISHING

SeptoPolisher

SILICONE POLISHERS FOR COMPOSITES, GLASS IONOMERS AND AMALGAMS

INDICATIONS

SeptoPolisher offers a shortened clinical process when working with silicone polishers.

Depending on the pressure applied, it is possible to have both finishing and polishing effects with just one tool. Applying more pressure intensifies the effectiveness of the finishing phase. To achieve the desired polishing quality use less force, which results in a smooth and bright enamel like surface.

FEATURES AND BENEFITS

- One silicone polisher is enough to achieve the desired finishing and polishing of the restoration, saving time and abrasive materials during the procedure.
- High resistance polishers can be used several times, suitable for sterilisation.
- Available in 3 shapes: cone, disc, cup.



PACKAGING

DESCRIPTION

SeptoPolisher* Assorted Refill:
8 pcs. + SeptoPlus Mandrel (3x cone, 3x disc, 2x cup)

SeptoPolisher* Refill:
8 pcs. cones + SeptoPlus Mandrel

DESCRIPTION

SeptoPolisher* Refill:
8 pcs. discs + SeptoPlus Mandrel

SeptoPolisher* Refill: 8 pcs. cups + SeptoPlus Mandrel

SeptoPlus Mandrel: 1 pcs.

*SeptoPlus Mandrel required

SeptoPolisher

Finishing & Polishing



FINISHING AND POLISHING

SeptoFipo Strip

POLYESTER STRIPS, ALUMINIUM OXIDE COATED


INDICATIONS

SeptoFipo Strips are indicated for finishing and polishing of the proximal faces of composite restorations.

Use the neutral centre to insert the strip into the proximal space. Start the restoration with the medium grit (gray) side and then polish with the fine (white colour) side.

FEATURES AND BENEFITS

- Medium and fine grits separated by a neutral space for an easier insertion through the contact point area, without harming it
- Available in 2,5 mm and 4,0 mm widths

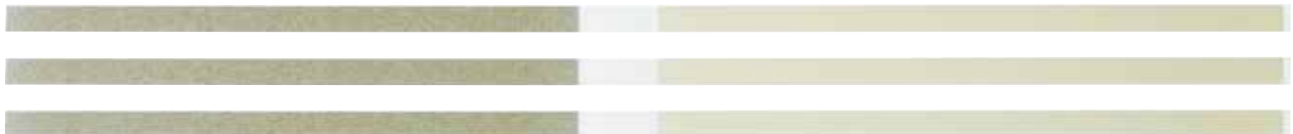

PACKAGING

DESCRIPTION

SeptoFipo Strips: 100 strips (2,5 mm)

SeptoFipo Strips: 100 strips (4 mm)

SeptoFipo Strip

Finishing & Polishing



FINISHING AND POLISHING

SeptoDiamond Strip

STAINLESS STEEL STRIPS; DIAMOND COATED

INDICATIONS

Diamond Strip is a diamond coated steel strip indicated for finishing of filling materials and orthodontic stripping.

To achieve the best results it is recommended to use both strips, starting with the medium grit and concluding with the fine grit.

FEATURES AND BENEFITS

- Excellent finishing on restorations
- High performance on orthodontic stripping
- High durability
- Available in fine (yellow) and medium (red) grits – 2,5mm width.



PACKAGING

DESCRIPTION


SeptoDiamond Strips: 12 strips 2,5mm (fine)

SeptoDiamond Strips: 12 strips 2,5mm (medium)

SeptoDiamond Strips: 12 strips 2,5mm (medium & fine)

SeptoDiamond Strip

Finishing & Polishing

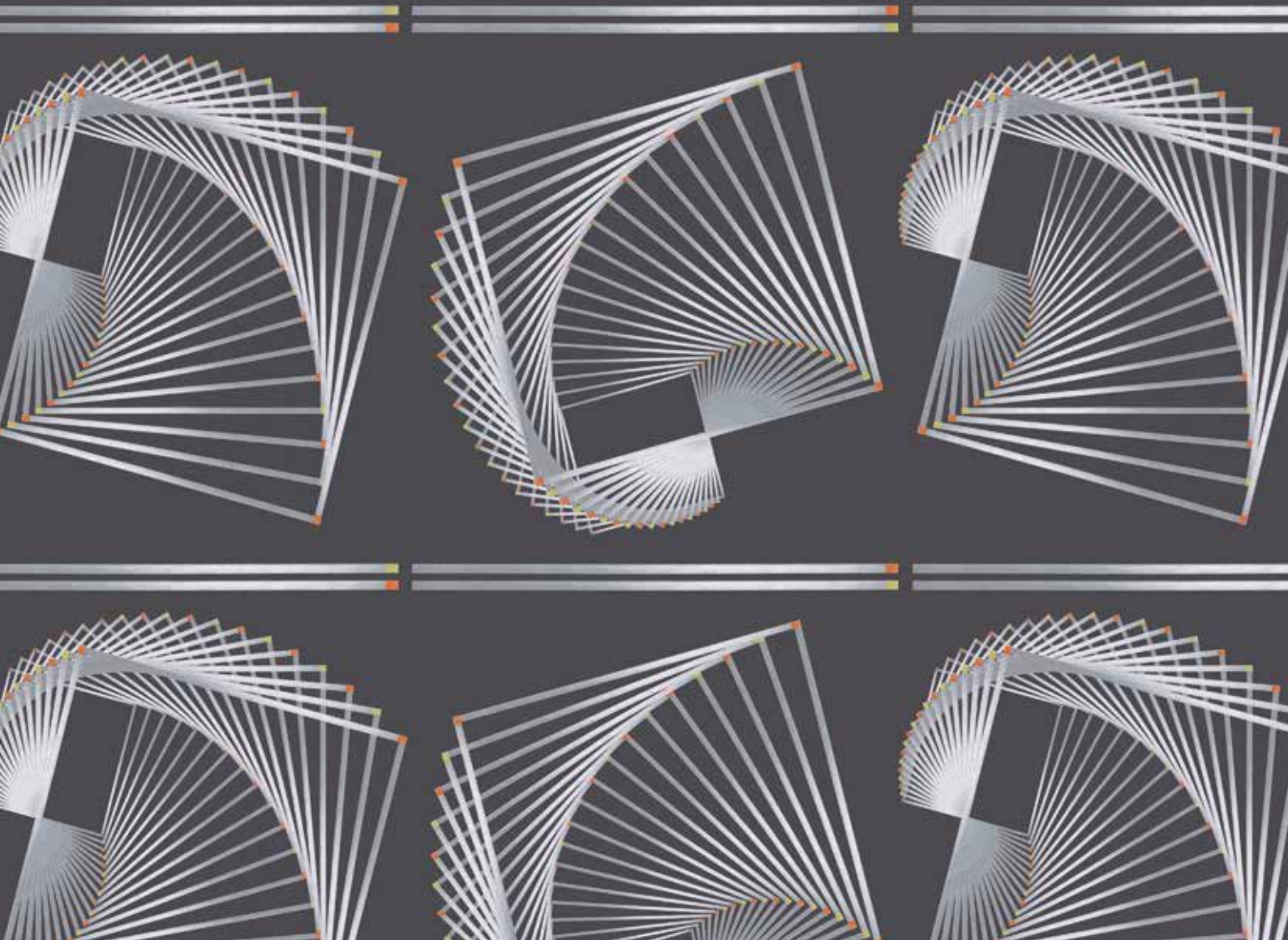

FINISHING AND POLISHING

SeptoDiamond Paste

DIAMOND POLISHING PASTE

INDICATIONS

SeptoDiamond Paste is a diamond polishing paste designed for final polishing and obtaining the highest quality lustre of all direct and indirect restorations. Provides the best possible gloss for enamel, ceramics, composites, amalgams and metal surfaces. Recommended for use with SeptoWheels PX discs.

FEATURES AND BENEFITS

- High gloss and an excellent natural brightness on all restorative materials and enamel
- Possible to use in both dental clinics and laboratories
- Extra-fine grit (1 to 2 microns)
- Hydrosoluble, so residues are easy to remove.



PACKAGING

DESCRIPTION

SeptoDiamond Paste: 2g syringe

SeptoDiamond Paste

Finishing & Polishing



MATRICES

SeptoMatrix Cervical

TRANSPARENT MATRICES SYSTEM FOR CLASS V RESTORATIONS

INDICATIONS

SeptoMatrix Cervical is an excellent tool, facilitating the predictable modeling of direct composite restorations. Experienced practitioners are extremely pragmatic in their choices and therefore appreciate the highest quality of accessories and instruments. The set of matrices includes five flexible shapes in different sizes. They provide excellent working comfort, reduce working time and guarantee perfect adaptation near to gingival margins.



FEATURES AND BENEFITS

- Due to multiple sizes, they can be easily adapted to all class V restorations
- Composites can be light cured through the transparent matrix while it is held under pressure, obtaining a smooth surface that will require less adjustment and just a simple polishing
- Does not adhere to composites or glass ionomers
- Suitable for sterilisation.



PACKAGING

DESCRIPTION

SeptoMatrix Cervical:
2 instruments + 10 matrices (2x Extra-Large, 2x Large, 2x, Medium, 2x Small, 2x Extra-Small)

SeptoMatrix Cervical Assorted Refill:
50 matrices (10x Extra-Large, 10x Large, 10x Medium, 10x Small, 10x Extra-Small)

DESCRIPTION

SeptoMatrix Cervical Refill: 50 matrices Extra-Large

SeptoMatrix Cervical Refill: 50 matrices Large

SeptoMatrix Cervical Refill: 50 matrices Medium

SeptoMatrix Cervical Refill: 50 matrices Small

SeptoMatrix Cervical Refill: 50 matrices Extra-Small

SeptoMatrix Cervical

Matrices



MATRICES

SeptoMatrix Sectional

PREFORMED SECTIONAL MATRICES SYSTEM WITH REINFORCED RINGS FOR CLASS II RESTORATIONS

INDICATIONS

The SeptoMatrix Sectional system is indicated for restorations of class II cavities. It provides optimal conditions for precise reconstruction of contact points and provides a natural tooth anatomy in proximal areas. The rings are equipped with triangular silicone tips for perfect adaptation of the fit. Strengthening of the rings guarantee controlled forces during separation – hard ring (blue) or soft ring (pink) – and greater stability during the reconstruction procedure with additional increased durability of the rings.

FEATURES AND BENEFITS

- Reinforcement made of durable material.
- Unique design: provides stability, enables safe opening with the help of forceps
- Easy, quick and comfortable placing by a ring system, which replace the typical matrix holders
- Sectional matrices available in 3 different sizes for better adaptation to all clinical situations
- Rings with silicone tips suitable for sterilisation.



PACKAGING

DESCRIPTION

SeptoMatrix Sectional Assorted Kit:
100 matrices (30x Large, 40x Medium, 30x Small)
+ 2 rings (hard and soft) + 14 triangular silicone tips

SeptoMatrix Sectional Assorted Kit:
50 matrices (15x Large, 20x Medium, 15x Small)
+ 2 rings (hard and soft) + 8 triangular silicone tips

SeptoMatrix Sectional Refill:
1 ring (hard) + 8 triangular silicone tips

DESCRIPTION

SeptoMatrix Sectional Refill:
1 ring (soft) + 8 triangular silicone tips

SeptoMatrix Sectional Assorted Refill: 50 matrices
(15x Large, 20x Medium, 15x Small)

SeptoMatrix Sectional Refill: 50 matrices Large

SeptoMatrix Sectional Refill: 50 matrices Medium

SeptoMatrix Sectional Refill: 50 matrices Small

SeptoMatrix Sectional triangular silicone tips: 50 pcs.



SeptoMatrix Sectional
Matrices

WEDGES

SeptoWedges

ANATOMIC WOODEN WEDGES

INDICATIONS

One of the crucial requirements of the direct filling procedure is the reconstruction of a proper anatomical shape. This, of course, involves the need to use methods and instruments that increase the effectiveness and predictability of the given treatment. The high-quality wooden wedges are one of such important elements which are always included in the dentist's work flow protocol.

FEATURES AND BENEFITS

- Anatomically shaped for a better adjustment to the interproximal spaces and to separate adjacent teeth
- Available in 5 sizes codified by colours



PACKAGING

DESCRIPTION
SeptoWedges Assorted: 100 pcs. (30x yellow, 20x turquoise, 20x red, 20x violet, 10x natural)
SeptoWedges Refill: 100 pcs. – yellow
SeptoWedges Refill: 100 pcs. – turquoise
SeptoWedges Refill: 100 pcs. – red
SeptoWedges Refill: 100 pcs. – violet
SeptoWedges Refill: 100 pcs. – natural

No.	Height	Width	Length
1	2,2 mm	1,4 mm	12 mm
2	2,2 mm	1,4 mm	13 mm
3	2,2 mm	1,6 mm	14 mm
4	2,2 mm	1,6 mm	15 mm
5	2,5 mm	2,0 mm	16 mm

SeptoWedges
Wedges



ISOLATION

SeptoTape

TAPE FOR DENTAL ISOLATION

INDICATIONS

SeptoTape is indicated for tooth isolation during acid conditioning as well as for the application of adhesives, composites and other processes.

Made of inert and flexible material, the tape fits around the tooth, protecting it against undesirable contamination during the restorative procedure performed on the adjacent tooth.

FEATURES AND BENEFITS

- Protects the isolated teeth against undesirable aggressions
- Resistant to acids, adhesives and other chemical agents
- Very flexible and thin to enable easy insertion into the interproximal area
- Exclusively designed for dental use




PACKAGING

DESCRIPTION

SeptoTape: 5 m tape for dental isolation

SeptoTape
Isolation

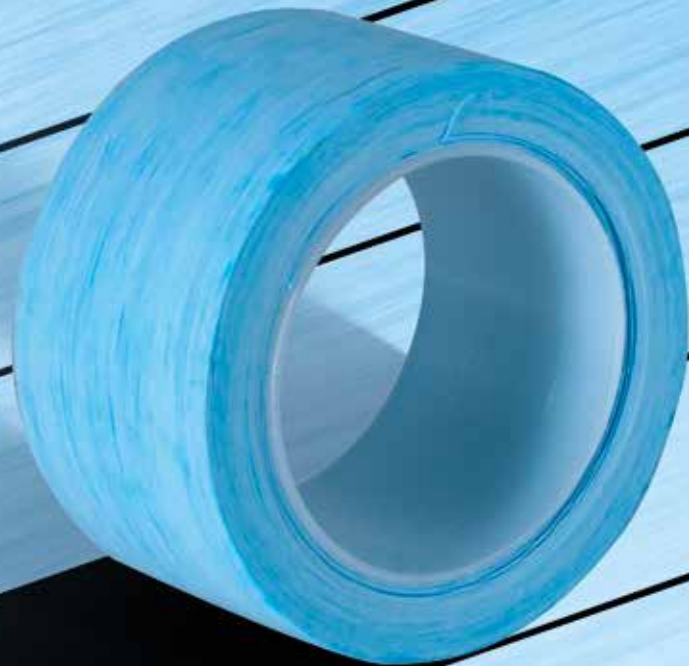

PROPHYLAXIS

SeptoProphy Strip

NON-ABRASIVE STEEL STRIPS FOR INTERPROXIMAL TARTAR REMOVAL


INDICATIONS

SeptoProphy Strips are indicated for removing dental plaque and tartar.

SeptoProphy Strips are an indispensable aid during prophylaxis procedures. With circular micro-blades located around the holes in one face of the strip it provides a powerful and precise action against the calculus, thus, preserving the integrity of enamel and restorative materials.

FEATURES AND BENEFITS

- Easy, fast and safe removal of tartar without damaging the dental enamel
- Non-abrasive, preserves the enamel, restorative materials and prosthetic elements
- Circular micro-blades provide more precision and quickens removal of dental plaque and tartar
- Easy access to the interproximal area due to its thickness of 0,05 mm

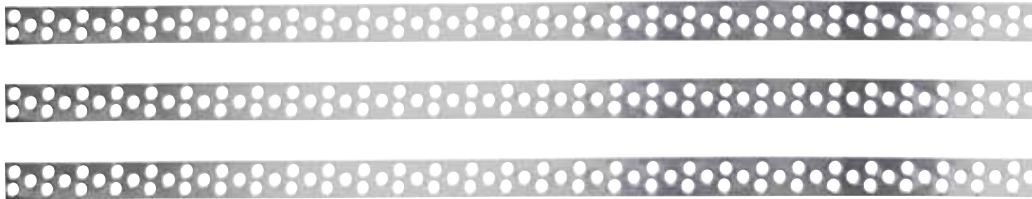

PACKAGING

DESCRIPTION

SeptoProphy Strips: 10 strips



SeptoProphy Strips
Prophylaxis

www.septoaccessories.com



Septodont Ltd

Units R & S - Orchard Business Centre

St Barnabas Close - Maidstone - Kent - ME16 0JZ

Tel.: +44 (0) 1622 695520

www.septodont.co.uk