


GenENDO Motor

Quick Guide




Motor overview

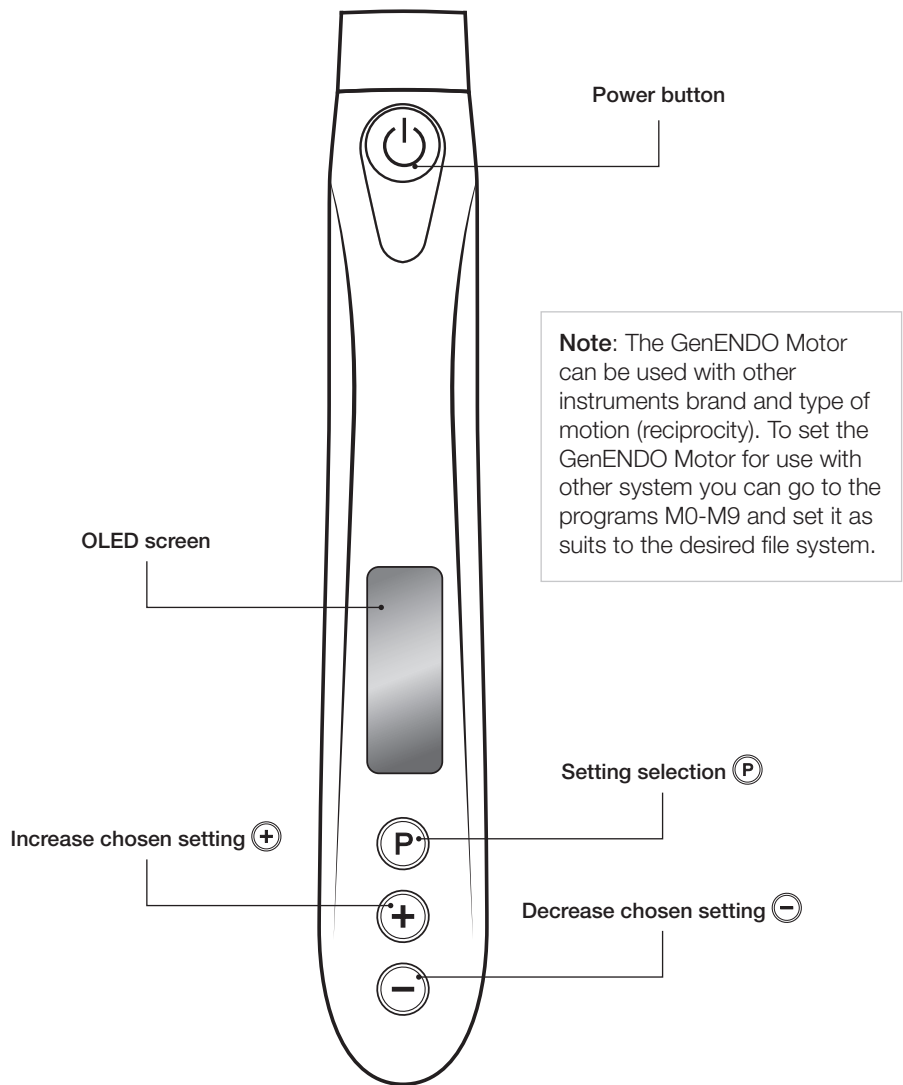

Turn on the motor

Press the power button to turn on the GenENDO Motor.

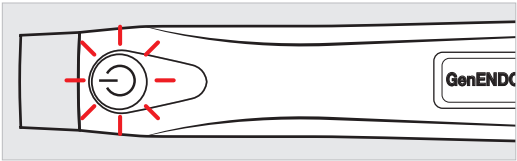

Motor presets

The GenENDO Motor is provided with presets for GenENDO continuous rotation instruments.

GenENDO Revo-S+ (SC1, SC2, SU) Glider & Remover settings for continuous rotation. Use the (P) button to switch between instrument settings.


GenENDO Revo-S+ (SC1, SC2, SU) continuous rotation settings

- Speed: 400 rpm
- Torque: 2,5 N.cm

Warning


The motor includes presets for both *continuous rotation (CR)* and *reciprocating rotation (RR)*.

Be sure to select the continuous rotation mode Revo-S+CR for safe and reliable operation.

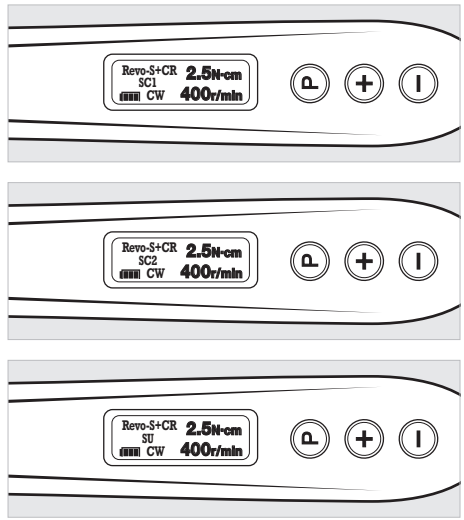


GenENDO Glider continuous rotation settings

- Speed: 350 rpm
- Torque: 1,2 N.cm

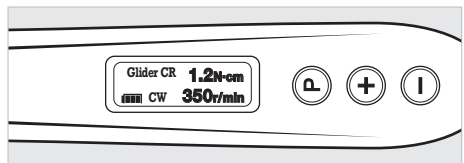

GenENDO Remover continuous rotation settings



- Speed: 800 rpm
- Torque: 2,5 N.cm

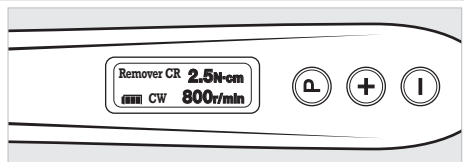

Turn off the motor

To shut down:

Hold the (P) button, then press the power button.

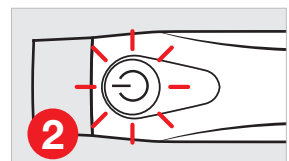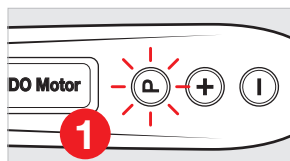

Maintenance and cleaning recommendations


Contra-angle

It is recommended to clean and disinfect the contra-angle **before the first use, and after each use.**

Cleaning and disinfection instructions


- Using the connector, clean and disinfect contra-angle with disinfectant of neutral pH value
- After disinfection, lubricate it with specific cleaning oil
- Finally, sterilize it under high temperature and high pressure (134 °C, 2.0 bar~2.3 bar (0.20 MPa~0.23 MPa))

The contra-angle needs to be lubricated after cleaning and disinfection, before sterilization.

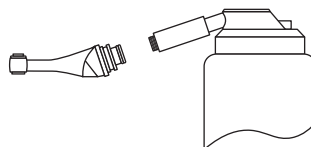

Battery charging

- When it stops charging, the screen battery indicator stops flashing.
- After charging, unplug the power adapter.
- The GenENDO Motor must be used with the original power adapter (5V 1A).

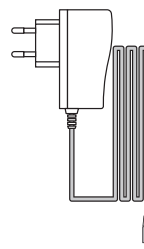

GenENDO Motor settings

Instrument	Motion	Torque (Ncm)	Speed (rpm)
GenENDO Revo-S+ CR (SC1 SC2 SU)	Continuous Rotation	2.5	400
GenENDO Glider	Continuous Rotation	1.2	350
GenENDO Remover	Continuous Rotation	2.5	800

Septodont

58 rue du Pont de Créteil - 94100 Saint-Maur-des-Fossés - France

www.septodont.com

Follow us on social media channels:

